Uddevalla, 12 mars 2020

## LifeClean - Coronavirus

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### Testresultat EN 14476:2013+A2:2019

Test norm	Test type	Organism	Log reduction required	Log reduction achieved	Laboratory	Clean or dirty	Contact time	LC Std
EN 14476:2013+ A2:2019 (Phase 2 Step1)	Quantitative suspension test, Screening	Feline coronavirus, Strain Munich	4 log <sub>10</sub>	>4.25 log <sub>10</sub>	MSL, Microbiological Solutions Ltd, United Kingdom	Clean	1 min	200

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Test identification Reference: J001375

Reduced Screening test based on - BS EN 14476:2013+A2:2019

## Test Result Summary

The test product shown a log reduction of 4.25 when tested under the conditions stipulated in this report with a contact time of 1 minute.

The test product shown a log reduction of 4.33 when tested under the conditions stipulated in this report with a contact time of 2 minutes.

The members of the family Coronaviridae are enveloped and have a positive sense RNA genome. Coronaviruses have a distinct morphology with an outer 'corona' of embedded envelope spikes. These viruses cause a broad spectrum of animal and human disease.

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Tel: 0522 506 475

E-post: [rahma.wehelie@lifeclean.se](mailto:rahma.wehelie@lifeclean.se)

**From Annex B in EN 14476:  
Examples of viruses which may contaminate human medical instruments,  
hands, surfaces (*Enveloped viruses are in bold*)**

NOTE This list is not exhaustive.

Blood

Enterovirus	<b>Hepatitis C virus (HCV)</b>
<b>Filoviridae</b>	<b>Hepatitis Delta virus (HDV)</b>
<b>Flavivirus</b>	<b>Human Immunodeficiency Virus (HIV)</b>
<b>Herpesviridae</b>	<b>Human T Cell Leukemia Virus (HTLV)</b>
Hepatitis A Virus (HAV)	Parvovirus B 19
<b>Hepatitis B virus (HBV)</b>	

Respiratory tract

Adenovirus (Mast-)	<b>Influenza Virus</b>
<b>Coronavirus</b>	<b>Paramyxoviridae</b>
Enterovirus	Rhinovirus
<b>Herpesviridae</b>	<b>Rubella Virus</b>

Neural tissue, ear & nose, eye

Adenovirus (Mast-)	<b>Human Immunodeficiency Virus (HIV)</b>
Enterovirus	Polyomavirus
<b>Herpesviridae</b>	<b>Rabies Virus</b>
<b>Measles Virus</b>	<b>Rubella Virus</b>

Gastro-intestinal

Adenovirus(Mast-)	Enterovirus
Caliciviridae	Hepatitis A Virus (HAV)
<b>Coronavirus</b>	Hepatitis E Virus (HEV)
Astrovirus	Rotavirus

Skin, breast and/or milk

Enterovirus	<b>Human T Cell Leukemia Virus (HTLV)</b>
<b>Herpesviridae</b>	Papillomavirus
<b>Human Immunodeficiency Virus (HIV)</b>	<b>Poxviridae</b>

Spleen and lymph nodes (see also „Blood“)

**Human T Cell Leukemia Virus (HTLV)**  
**Human Immunodeficiency Virus (HIV)**

Dental procedure

Adenovirus(Mast-)	<b>Hepatitis C Virus (HCV)</b>
Enterovirus	<b>Hepatitis Delta Virus (HDV)</b>
<b>Herpesviridae</b>	<b>Human Immunodeficiency Virus (HIV)</b>
<b>Hepatitis B virus (HBV)</b>	

Urogenital tract

<b>Hepatitis B Virus (HBV)</b>	<b>Human T Cell Leukemia Virus (HTLV)</b>
<b>Herpesviridae</b>	Papillomavirus
<b>Human Immunodeficiency Virus (HIV)</b>	Polyomavirus